リメイク設定方法
NobelProcera® Product

NobelProcera®

Product

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NobelProcera® Product

Content

•Information 03

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- NobelProcera® FCZ Crown 06
- NobelProcera® FCZ Bridge 08
- NobelProcera® FCZ Implant Bridge 11
- NobelProcera® Implant Bridge Zirconia 13

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•Contact support 24

リメイク設定方法 NobelProcera® Product

information

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DTX Studio Lab

NobelProcera Software Remake Order - NobelBiocare

Place your order
Order or export your designed products.
START

Case design

Model scan

Prescription form

Details

Activity

Restoration details

Production method
Nobel Biocare production

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Order for NobelProcera Software Remake Order
Patient of NobelBiocare

Issued by: _____ Invoice Reference: _____ 直接入力します

Ship to: _____
Marketing: Grand Central Tower 2-16-4 Konan B1
1400031 TOKYO Japan

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The screenshot shows the DTX Studio Lab interface. The main window is titled 'NobelProcera Software Remake Order - NobelBiocare'. On the left, there is a sidebar with a list of cases. The main area shows a detailed view of an order for 'Order GIQW73'. A callout box highlights the 'Order GIQW73' label and the 'Nobel Biocare production items' section, which includes the item name 'NobelProcera® Zr Implant Bridge 3uni...', the status 'In production', and the article number '83281'. The callout also contains the text '新しいオーダーのファイルネーム' (New order file name).

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DTX Studio Lab

NobelProcera Software Remake Order - NobelBiocare

Place your order
Order or export your designed products.
START

Case design

Model scan

Prescription form

Details Activity

Restoration details

Tooth 26 Full-arch crown HT ML Zirconia Allight

Patient Case ID
Overdate 14 OCT 2022

Requested by NobelBiocare

Place your order
Order or export your designed products.
START

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The screenshot shows a web application window titled "Create order" with a "Cancel" button in the top right. The main heading is "Order for NobelProcera Software Remake Order" and the sub-heading is "Patient of NobelBiocare".

Fields for "Issued by" and "Invoice Reference" are present. The "Ship to" field contains the address: "Marketing, Grand Central Tower 2-16-4 Konan B/F, 1400001 TOKYO, Japan". A link "Add a production remark" is visible next to it.

The "Order list (1) article" section contains a table with the following data:

TOOTH 26	NobelProcera HT Multi Layered FCZ Crown ALight	production	1x	JPY 0.00	...
	Article number 73446				
				Total price	JPY 0.00

An "Order now" button with a right-pointing arrow is located at the bottom of the window.

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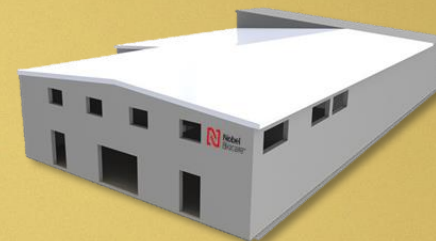
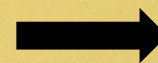
The collage displays five pages from the 'NobelProcera Software / Remake Order Manual'. The pages are:

- Top Left:** 'NobelProceraソフトウェア/リメイクオーダー マニュアル' (NobelProcera Software / Remake Order Manual) - Introduction and overview.
- Top Middle:** 'リメイクオーダーの準備' (Remake Order Preparation) - Details on preparing the remake order.
- Top Right:** 'リメイクオーダーの作成' (Remake Order Creation) - Step-by-step guide to creating the remake order in the software.
- Middle Left:** 'リメイクオーダーの入力' (Remake Order Input) - Instructions on how to input the remake order details.
- Middle Right:** 'マウア(米国)製造におけるNobelProcera製品のリメイクオーダー方法' (Remake Order Method for NobelProcera Products Manufactured in the USA) - Specific instructions for the US manufacturing process.

Each page includes the Nobel Biocare logo, a title, and detailed text with images of dental models and software interface elements. A red banner at the bottom of each page contains contact information for Nobel Biocare Japan K.K.

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